



MX3

Hydration Testing System
First Time Set Up



Hydration Testing System



More information at mx3diagnostics.com/resources

1 Register for an MX3 Account on the MX3 Portal (portal.mx3diagnostics.com).

2 When prompted, enter your 4-character MX3 LAB serial number. To find your serial number:



Turn on the MX3 LAB.



Select **INFO** to display system info, including device serial number.

3 Download the MX3 App from the iOS App Store or Google Play Store.



4 Large Organization/Professional Team:

Proceed to your organization subdomain in the MX3 Portal to set up teams, trainers, coaches and athletes. Log in to MX3 App to make a measurement.



Small Organization/Small Team:

Proceed to the MX3 Portal to add athletes for your organization, or log in directly to the MX3 App to add athletes and make a measurement.



Individual: Log in to the MX3 App to make a measurement.